

NuvōMed®
Have a Question?



Expert Support Available



CALL US!
1-(888)-738-6886

www.nuvomed-hearingaids.com/support



BACK

NuvōMed®
Quick Start
User Guide

DigiEars | MAX

Complete In-Canal
Hearing Aid with
Charging Case



COVER

Whats in the Box?

- Digital Hearing Aid x 1 Pair
- Ear Tips x 3 Pairs
(2pcs each for small, medium & large)
- Charging Case x 1
- USB Cable x 1
- Manual x 1
- Cleaning Tool x 1
- Carrying Pouch x 1




Storage

To ensure the longevity of your hearing aids. Safely store the device in the bag provided.



Maintenance & Cleaning

Remove the ear tips from the hearing aids. Use the cleaning brush to remove wax, dirt or any debris from both the ear tips and the device itself.

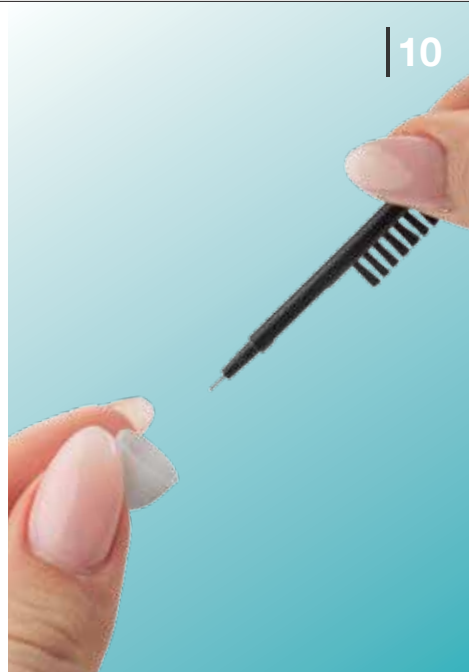

Clean

Make sure your ears are free of any dirt or debris.



Charge

Charge the case for 2.5 hours before use.



Battery

To check the remaining charge in the case & the hearing aids, press the battery button on the side of the case.




Fitting

Gently place the hearing aids in your ear.



Ear-Tips

For best use choose the ear tips with a comfortable & snug fit for your ear.

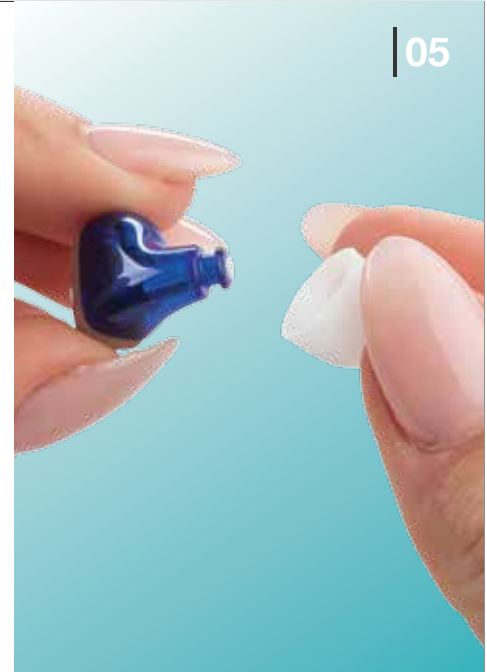

Power ON / OFF

To power On/Off your hearing aids hold the power button on each hearing aid.

06



Setting

Set each to lowest volume and adjust up as needed by pressing the power button on each hearing aid.

07

